**TOPRO  
TAURUS E**



**TOPRO**

**NO Bruksanvisning 2 - 7**  
**EN User manual 8 - 13**  
**DE Bedienungsanleitung 14 - 19**

**[www.topro.no](http://www.topro.no)**

**CE**










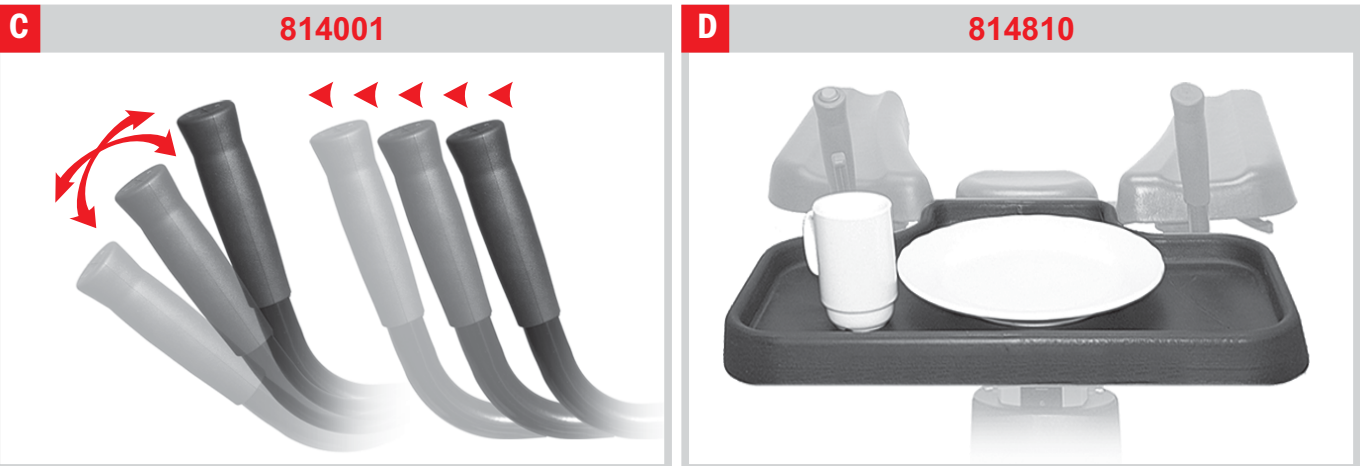
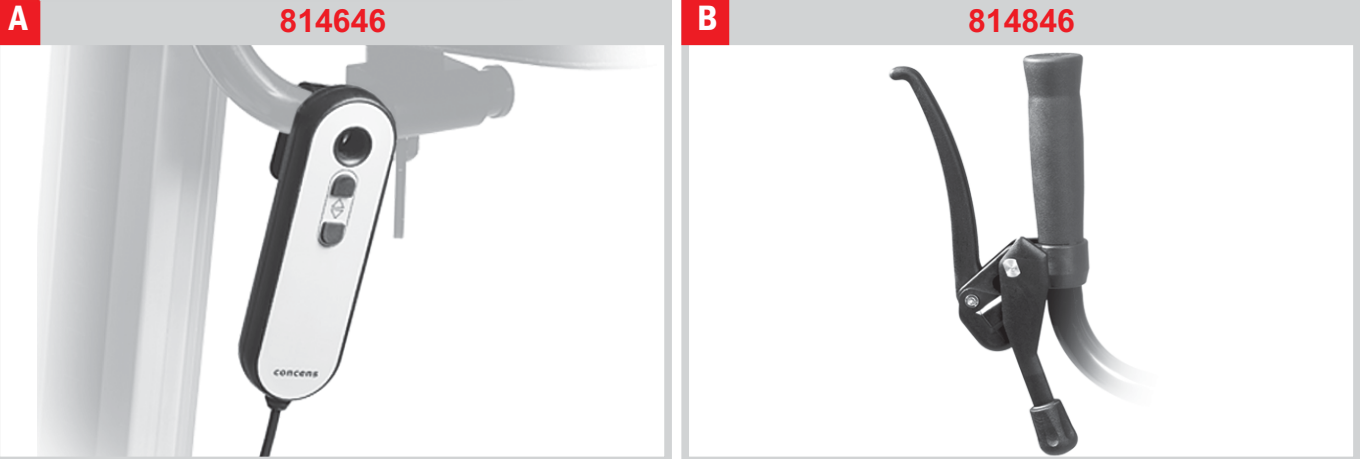
# TOPRO TAURUS E

**BASIC**  
**PREMIUM**

**814790**  
**814789**

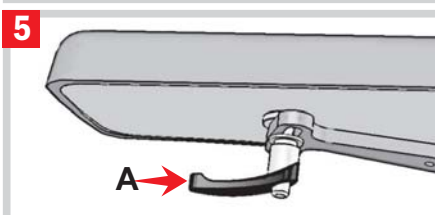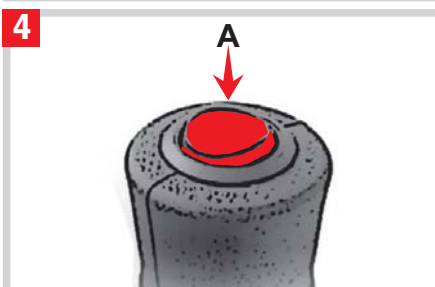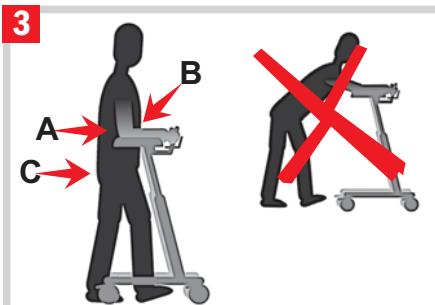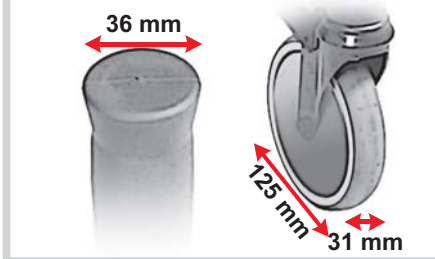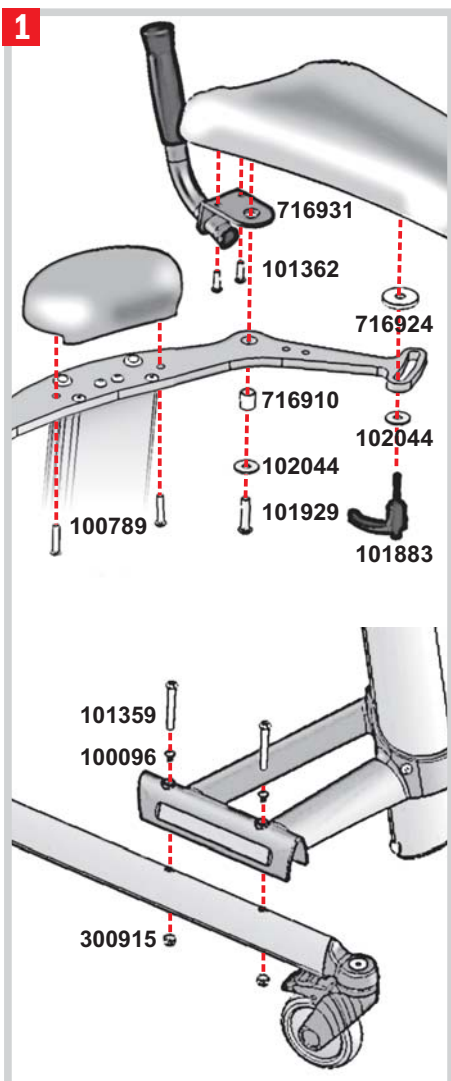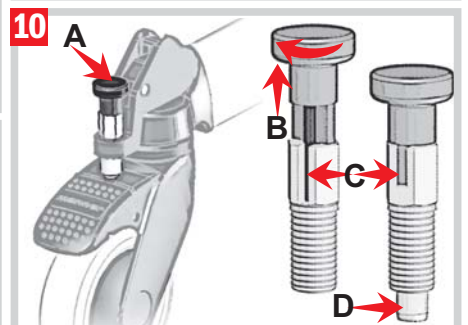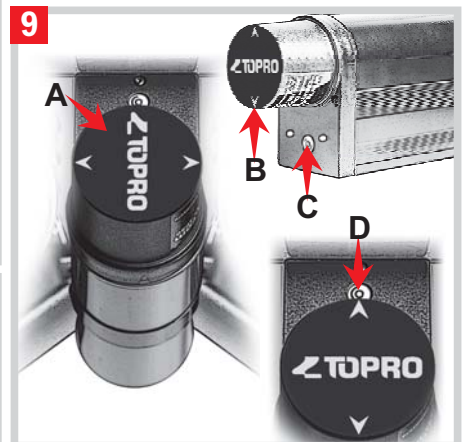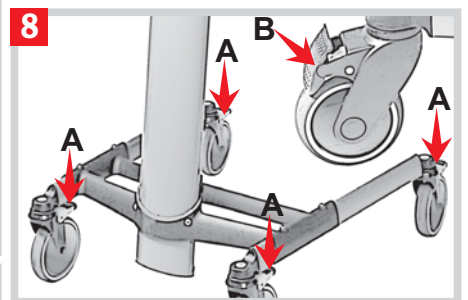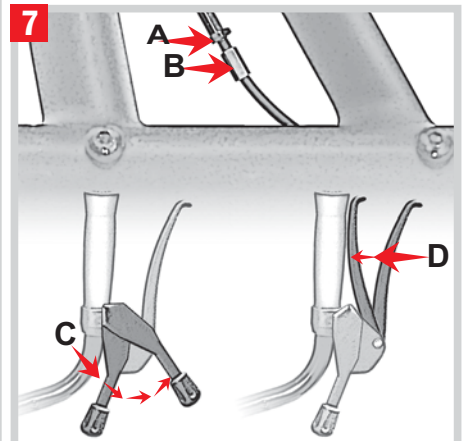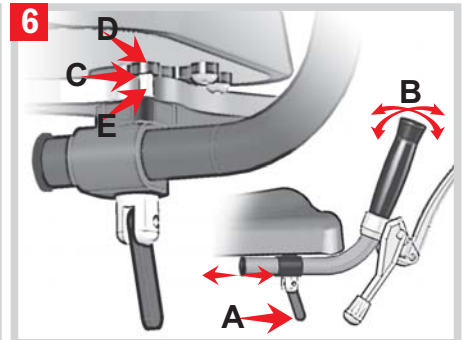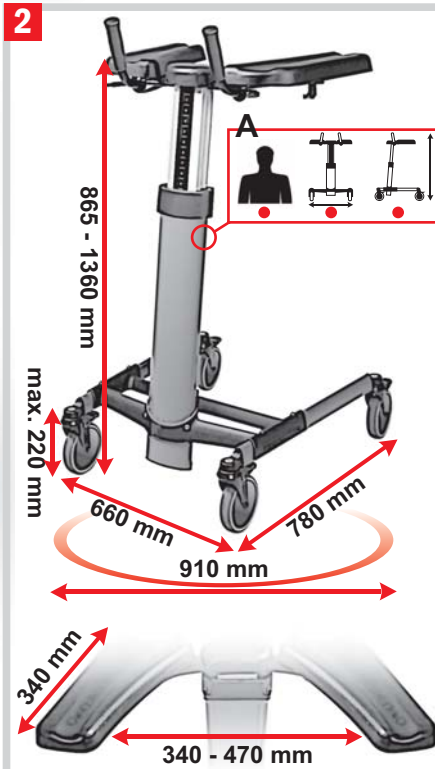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## Welcome as a user of a TOPRO PRODUCT

Every TOPRO product is developed, designed and made in Norway. Advanced manufacturing equipment, quality and environmental security gives great accuracy and maximum quality in all levels of production. The product is declared to be in conformity with the Medical Device Directive 93/42/EEC.

### Guarantee

TOPRO TAURUS is guaranteed free for faults and defects for 2 years. An exception is if the damage is caused by incorrect use or parts that are exposed to natural wear and tear (such as e.g. brakes, wheels and handles). For repairs in the guarantee period, please refer to your local mobility shop or dealer. The guarantee is void if unauthorised spare parts and accessories have been or are being used on the product.

The red numbered squares below refer to the numbered illustrations on the inside of the cover.

## 1 Assembling and adjusting

TOPRO TAURUS E Basic needs assembling. TOPRO TAURUS E Premium is delivered completely assembled.

Adjust the forearm supports and handles, see figure 5. The battery must be charged, see figure 9.

## 2 Measurements

A The label on the rear side of the column shows the product name/model and

- Maximum weight of user
- Maximum width of walker
- Maximum height of walker

Technical data	
Distance between front and rear wheel	780 mm
Width between front wheels	660 mm
Height of forearm supports	865 – 1360 mm
Distance between forearm supports	340 – 470 mm
Turning width	910 mm
Hand grip length	90 mm
Hand grip width	36 mm
Grip distance driving brake	74 mm
Wheel width	31 mm
Wheel diameter	125 mm
Weight walking table (w/o accessories)	22 kg
Maximum weight of user	150 kg
Material	
Frame	Aluminium
Push handles	PVC
Brake handles	PA6
Forearm supports	PU-foam
<b>For technical details on the electrical components, see the last section of the user manual</b>	

## TOPRO TAURUS E is delivered with two levels of equipment

**Basic (814790)** Adjustable arm pads and brakes on all wheels. Delivered partly assembled.

**Premium (814789)** In addition to adjustable arm pads and brakes on all wheels it has adjustable handles and a driving and parking brake on the handle. The back wheels have a lock for swing (directional block). Delivered completely assembled.

### Intended use

The TOPRO TAURUS is designed and approved to be used indoors on plain foundation. The walker safely supports users with poor balance and/or poor mobility. The TOPRO TAURUS E Premium can also be used as a raising support.

## 3 Important to know about safety and use

- The walker must be adjusted to the recommended height for right support and safe walking: Stand with relaxed shoulders and have your elbow at a 90 degree angle **A**. Hold the walker as close to your

body to maintain the correct support and a safe walk **B**. The easiest way to keep your balance is to walk straight, look forward and hold your backside in **C**.

- Do not push the walker too far from your body as this can cause unnecessary strain and you could fall.
- Do not push the walker forward if the front wheels have stopped at an obstacle. The walker may tilt and cause an accident.
- Max user weight is 150 kg. Max user height 2 m.
- Do not let children play with the walker.
- Some parts of the walker might feel hot or cold when exposed to extreme temperatures.
- Be aware: TOPRO TAURUS E must not be used in a damp environment.

#### **4** Height Adjustment

Press the button **A** on top of the handle to the left or to the right to regulate the height.

#### **5** Adjusting the forearm support

The forearm supports are adjusted individually. Unscrew the lock underneath the forearm support **A** until the cushion is moveable. Place at the required position and tighten the screw. **NB** If the lock sticks out, it can be dangerous. Pull the lever down and rotate it so that it doesn't stick out.

#### **6** Angle adjustment and twisting the handle, TOPRO TAURUS E Premium

Unscrew the screw **A** until you can move the handle. Adjust the handle **B** to the required angle, length and twist and tighten the screw. **NB** The screw must not be screwed too far out as the parts will fall from each other. If this happens, then put the pin **C** in the hole **D** and **E**. It is not possible to pull the handles all the way out; there are physical stops both ways.

See figure 1 for more detailed instructions.

#### **7** Brakes, TOPRO TAURUS E Premium

##### **Adjusting the brakes**

Unscrew the nut **A**. Tighten / loosen the brake wire with the adjusting screw **B**. Make sure the brakes work properly and don't allow the brake blocks to touch the wheel when brakes are off. Adjust the brakes evenly on both sides.

##### **Using the parking brakes**

The brake lever **C** works on both rear wheels simultaneously. Push the brake lever forward. It has some resistance before it stops and locks both the rear wheels. Pull the brake lever backwards to release the rear wheels. Inspect the brakes frequently to make sure they work properly.

##### **Using the driving brakes**

The brake lever **D** works on both rear wheels simultaneously. Pull the brake lever towards you to stop the rear wheels. When the brake lever is released, the rear wheels are released. Inspect the brakes frequently by following the information above.

#### **8** Using the foot brakes

Step on the foot brakes **A** on the rear wheels or when required on all wheels until you hear a clicking sound.

The wheels are now in a locked position **B**. To release the wheels, step on the back of the foot brake pedal so that it comes up again.

#### **9** Charging the battery

It is important to charge the battery on a regular basis to prolong the battery life. We recommend charging the battery every 24 hours. Regular charging will improve the performance of the battery. Please note: If the walker is not used over a long period of time, then the battery should be taken out and placed in the charger otherwise it will lose power and efficiency. It takes approximately 5-8 hours to fully charge the battery.

Charging: Connect the charger to the power grid without the battery. A yellow light will shine and a green light will flash. Hold the battery, turn it 45 degrees and lift it out of the control box of the walker **A**. Please check that the contact surfaces are clean. Put the battery in the battery charger; check that one of the arrows on the battery **B** points towards the white and blue mark/eye **C**. A green light will flash during charging. The light will show a constant green colour when the battery is fully loaded. To mount the battery on the walker again put the battery back in the control box on the column. Check that one arrow points towards the white and blue mark/eye and the other arrow points straight back **D**.

**Be aware: When the battery level is very low, it might only be possible to adjust the height of the walker down, not up. The battery has to be reset if this occurs. Take the battery out of the control box and press the red height adjustment button on the handle (see illustration 3) until you hear a beep.**

#### Different beep signals:

**Short “beep...beep”** from the control box every 24<sup>th</sup> second = low battery level. Charge the battery.

**Long “beeeeeep”** when the battery is put in the control box = good connection

**Short “beep...beep** when the Taurus is being raised and lowered = Low voltage. The battery should be replaced

**Long “beeeeeep”** when the Taurus is being raised and lowered = battery voltage lower than 17,6. It will only be possible to lower it, not raise it.

**Short “beep...beep...beep...beep”**= too high temperature inside the control box. Stop using until the control box has cooled down.

## 10 Locking the swing (directional block), TOPRO TAURUS E Premium

To be done with an index pin **A** on both wheels. Make sure the mounting bracket for the foot brake is pointing backwards.

#### This is how the index pin works:

Locking: Twist the upper part of the index pin **B** so that it fits into the track of the lower part **C**. By doing this the pin **D** will slide out and lock.

Releasing: Pull the upper part of the index pin **B** up and turn it 90 degrees. NB: Make sure it stays in the upper track. It will then lock in the upper position. The pin **D** slides in and the index pin is unlocked.

Please note: If there is no need to lock the swing but there is a risk that the user may unintentionally release the index pin, we recommend to remove it. This is done by screwing it out with an adjustable spanner.

## Material / Recycling

The walker is made of plastic coated aluminium pipe, plastic couplings and material of plastic and polyester. The forearm cushion is of polyurethane. Most parts can be recycled.

The battery contains NiMH (nickel). Do not dispose the battery in ordinary household waste, some communities offer recycling or collection of batteries. Contact your local government for disposal practices in your area.

## Cleaning / Disinfections

The walker can be cleaned with household detergents. It cannot be hosed because of the electric motor.

Cleaning with disinfections must be left to experienced personnel. To disinfect the walker, there must be used disinfection containing 70 - 80 % Ethanol. We advise against the use of disinfections containing chloride and phenol. The manufacturer cannot be incriminated if damages occur under disinfecting caused by wrong use or when cleaned by unauthorized personnel.

## Inspection / Maintenance / Re-use

The wheel bearings are sealed and maintenance-free. All the other parts do not need to be lubricated. It is recommended that general inspection and maintenance is carried out and that the items below are checked frequently: tubes and frame, handles and forearm supports, brakes, wheels and accessories. This also applies when the walker is to be prepared for re-use. Please see detailed instructions in this user manual about maintenance of the brakes and cleaning and disinfection of the walker.

## When something does not work

1 = possible reason

2 = action

### Brakes

1 Worn out / defect wheel.

2 Change the complete wheel.

1 Not braking effectively.

2 Adjust the brake wire.

1 Worn out / defect brake mechanism on wheel.

2 Change the entire back wheel section.

1 Worn out / defect brake handle.

2 Change the complete brake handle.

1 Worn out / damaged brake wire.

2 Change the brake wire.

### Height adjustment

1 Battery is flat/does not work.

2 Charge or change the battery. See point 9; Charging the battery

1 The height adjustment does not work.  
It only goes down, not up.

2 See point 9; Charging the battery

- |   |   |
|---|---|
| <ul style="list-style-type: none"> <li>1 The battery is not correctly mounted.</li> <li>1 Loose cable contact.</li> <li>1 Damage on the cable.</li> <li>1 Defect switch.</li> <li>1 Defect height adjusting panel (accessory).</li> </ul> | <ul style="list-style-type: none"> <li>2 Check that the battery is clean and correctly mounted.</li> <li>2 Check that all contacts are properly in place and tightly fitted.</li> <li>2 Check all cables and replace any damaged one(s).</li> <li>2 Change the switch.</li> <li>2 Change the height adjusting panel.</li> </ul> |
|---|---|

**In the event that all of the above has been checked and the product is still not operating correctly, then the cause may be damage to the control box or actuator. Please contact your supplier.**

### Accessories

For pictures and TOPRO article numbers see the inside of the cover. Accessories mounted on the walker may influence stability, we therefore advise you to use them carefully.

- |  |  |
|--|--|
| <ul style="list-style-type: none"> <li>A Height adjustment panel</li> <li>B Brake handle left side</li> <li>C Adjustable handles</li> <li>D Table</li> <li>E Stand for body fluid-boxes</li> </ul> | <ul style="list-style-type: none"> <li>F Small basket</li> <li>G Holder for oxygen</li> <li>H Footplate</li> <li>I Infusion stand</li> <li>J Wall mount for battery charger</li> </ul> |
|--|--|

The accessories are to be ordered separately to give the possibility to compose a TOPRO TAURUS that suits the individual users need. Contact your mobility shop, dealer or TOPRO for an updated overview of the accessories, or visit our homepage [www.topro.no](http://www.topro.no)

**In case of faults or questions please contact your mobility shop or dealer.**

### Technical details electrical components

Manufacturer Concens AS, Denmark

Controller box: C3con1-x

Actuator: Con 5

Battery: C3 NiMH 24 V – 1400 mAh

Battery charger: C3, Input voltage: 100-240VAC

Input frequency: 50/60Hz

Input current: Max 200mA (fuse 1A)

Output current: Max 400mA

Working ambient temperature for actuator: -5°C to +40°C

Working ambient temperature for controller and battery charger: 0°C to +40°C

Humidity: 20-80%

Delivery and storing ambient temperature for actuator, controller and battery charger: -20°C to +50°C

Humidity: 20-80%

### Guidance and manufacturer's declaration

#### Electromagnetic EMISSIONS for all ME EQUIPMENT and ME SYSTEMS

The TOPRO Taurus walking aid is intended for use in the electromagnetic environment specified below. The customer or the user of the TOPRO Taurus walking aid should assure that it is used in such an environment.		
Emissions test	Compliance	Electromagnetic environment – guidance
RF emissions CISPR 11	Group 1	The TOPRO Taurus walking aid uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	The TOPRO Taurus walking aid is suitable for use in all establishments, including domestic establishments and those directly connected to the public low voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Complies	
Voltage fluctuations/ flicker emissions IEC 61000-3-3	Complies	

**Guidance and manufacturer's declaration**  
**Electromagnetic IMMUNITY for all ME EQUIPMENT and ME SYSTEMS**

The TOPRO Taurus is intended for use in the electromagnetic environment specified below. The customer or the user of the TOPRO Taurus should assure that it is used in such an environment.			
IMMUNITY test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Electrostatic discharge (ESD) IEC 61000-4-2	± 6 kV contact ± 8 kV air		Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30 %.
Electrical fast transient/burst IEC 61000-4-4	± 2 kV for power supply lines ± 1 kV for input/output lines		Main power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	± 1 kV line(s) to line(s) ± 2 kV line(s) to earth		Main power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	<5 % UT (>95 % dip in UT) for 0,5 cycle 40 % UT (60 % dip in UT) for 5 cycles 70 % UT (30 % dip in UT) for 25 cycles <5 % UT (>95 % dip in UT) for 5 s		Main power quality should be that of a typical commercial or hospital environment. If the user of the TOPRO Taurus requires continued operation during power mains interruptions, it is recommended that the TOPRO Taurus be powered from an uninterruptible power supply or a battery.
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	3 A/m		Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.
NOTE: UT is the a.c. mains voltage prior to application of the test level.			

**Guidance and manufacturer's declaration**  
**Electromagnetic IMMUNITY for ME EQUIPMENT and ME SYSTEMS that are not LIFE-SUPPORTING**

The TOPRO Taurus is intended for use in the electromagnetic environment specified below. The customer or the user of the TOPRO Taurus should assure that it is used in such an environment.			
IMMUNITY test	IEC 60601 TEST LEVEL	Compliance level	Electromagnetic environment – guidance
Conducted RF IEC 61000-4-6	3 Vrms 150 kHz to 80 MHz	3 V	Portable and mobile RF communications equipment should be used no closer to any part of the TOPRO Taurus, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.  Recommended separation distance  $d=1,16\sqrt{P}$  $d=1,16\sqrt{P}$ 80 MHz – 800 MHz  $d=2,33\sqrt{P}$ 800 MHz – 2,5 GHz

Radiated RF IEC 61000-4-3	3 V/m 80 MHz to 2,5 GHz	3 V/m	<p>where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in metres (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, a should be less than the compliance level in each frequency range. b</p> <p>Interference may occur in the vicinity of equipment marked with the following symbol:</p> 
---------------------------	----------------------------	-------	---

NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies.  
 NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

a) Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the TOPRO Taurus is used exceeds the applicable RF compliance level above, the TOPRO Taurus should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the TOPRO Taurus.  
 b) Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.

**Recommended separation distances between portable and mobile RF communications equipment and the ME EQUIPMENT or ME SYSTEM for ME EQUIPMENT and ME SYSTEMS that are not LIFE-SUPPORTING**

The TOPRO Taurus is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled.  
 The customer or the user of the TOPRO Taurus can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the TOPRO Taurus as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter W	Separation distance according to frequency of transmitter m		
	150 kHz – 80 MHz $d=1,16\sqrt{P}$	80 MHz – 800 MHz $d=1,16\sqrt{P}$	800 MHz – 2,5 GHz $d=2,33\sqrt{P}$
0,01	0,12	0,12	0,23
0,1	0,37	0,37	0,74
1	1,16	1,16	2,33
10	3,67	3,67	7,37
100	11,6	11,6	23,3

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.  
 NOTE 1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.  
 NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

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