# TROJA 2G PREMIUM

... THE SECOND GENERATION

**Bruksanvisning**  
**User manual**  
**Bedienungsanleitung**  
**Manuel d'utilisation**  
**Handleiding**

## Bruksanvisning

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TOPRO art.nr. 814042  
HMS nr. 176156

## Get to know your TOPRO Troja 2G

Please refer to the illustration of the rollator on the inside of the cover

- 1. Ergonomical handles “TOPRO ERGO Grip”:**  
Shaped to provide support for your hands and help you stand from a seated position. Made from a soft and easy to clean PU-material. With an integrated attachment for the optional back support.
- 2. Enclosed loop brake handle:**  
Comfortable, secure and easy to use.
- 3. Bell:**  
Easy to use without having to take your hands off the handles.
- 4. Height adjustment with memory function:**  
Simple and secure adjustment of correct handle height for optimal ergonomics when walking.
- 5. Seat:**  
As comfortable as your garden chair.  
User weight: Max. 150 kg (M) / 125 kg (S).
- 6. Grey release strap is integrated into the seat:**  
Simply lift the strap to fold your rollator easily.
- 7. Side frames:**  
Features integrated attachments for various optional accessories.
- 8. Tilt function:**  
Makes it easier to navigate small obstacles i.e. kerbs and thresholds.
- 9. Brake wire**
- 10. Lock clamp:**  
Locks your rollator securely in its folded position to make handling and transportation easier.
- 11. Carrying handle and seat bracket:**  
Attachment point for the removable basket and the only recommended place to carry your rollator.
- 12. Detachable basket:**  
For simple and safe transportation of your small items and groceries etc, maximum weight limit 10 kilos.
- 13. Reflective panels:**  
Reflective panels in crucial spots on your rollator to make you more visible in the dark. 
- 14. Product markings:**  
Contains important information such as the name of the manufacturer, serial number, maximum load, measurements etc. (See also page 3).
- 15. Detachable and replaceable wheels, “TOPRO Quick”**
- 16. Interior braking system (IBS):**  
The brake blocks brake onto the rim instead of the surface of the wheel.
- 17. Edge guard:**  
Prevents the rear wheel from getting stuck on thresholds, chairs etc.

## Congratulations on your purchase of TOPRO Troja 2G

You have chosen a high quality product to help increase your mobility in everyday life. TOPRO has developed and manufactured this rollator in Norway, based on current research results, the highest standards of quality and security as well as our customers' demands of functionality.

The production of rollators at TOPRO is certified according to the standards of quality in ISO 9001 and the environmental standards of ISO 14001.

The TOPRO Troja 2G has been developed in accordance with the 93/42/EEC directive concerning medical equipment. It is tested and approved according to ISO 11199-2:2005.

On receipt of the goods please check your product, if you have any queries please contact your mobility shop or dealer immediately. The same applies if you have general questions about the product.

If you have problems reading this user manual an electronic version is available on [www.topro.co.uk](http://www.topro.co.uk)

We hope your TOPRO Troja 2G will improve your everyday life, giving you more mobility and personal freedom.

Sincerely,

The TOPRO logo consists of a stylized red 'Z' shape followed by the word 'TOPRO' in a bold, black, sans-serif font.

TOPRO Industri AS


## ⚠ Before using your TOPRO Troja 2G

- **Intended use:** TOPRO Troja 2G has been developed and approved for both indoor and outdoor use, to offer support and security for people with poor balance or walking ability. It is developed to be pushed, not pulled, and only to be used on plain surfaces. Outdoor the rollator is intended to be an aid for walking on roads and paths. It is not suitable in rough and/or steep terrain or inclinations, for example stairs. The product is not suitable for persons with low strength in arms, very poor balance, or considerable cognitive disabilities. The integrated seat provides an opportunity for the user to rest, and the fabric basket is designed for carrying items weighing up to a total of 10 kg.
- Take care not to get your fingers trapped when folding or unfolding your rollator.
- Before each use, check that your brakes are functioning correctly. Also check that your rollator locks into its unfolded position (you should hear a “click”) and the handles have been fastened properly.
- Always use both brake handles at the same time. This applies both to the driving brake and the parking brake. Before sitting down on the seat, make sure both parking brakes have been applied.
- The TOPRO Troja 2G should never be pushed while the parking brakes are on, or while the driving brakes are continuously applied.
- TOPRO Troja 2G has a maximum user weight of 150 kg (M) / 125 kg (S). It must not be used for the transportation of heavy goods or persons.
- For your own safety: Do not bend backwards or over to the sides when you are sitting on the seat. Do not tilt the rollator more than max 5° sideways to avoid risk of falling. Due to the same reason bags etc. must never be hung on the handlebars.
- Make children aware that the TOPRO Troja 2G rollator is not a toy.
- If the rollator is exposed to extreme temperatures, some parts may feel very cold or very hot.
- Do not wear very long or wide, flowing garments as they may get caught in the wheels of the rollator and be a potential hazard.
- For safety reasons it is prohibited to sit on the rollator during public transportation. Place your rollator securely. Use the parking brake.
- Do not use the rollator on stairs. Be careful when descending kerbs.
- Be aware that the rollator may roll faster than you walking downhill, which may put you in risk of falling.
- Do not alter the product as this might put your safety at risk and the guarantee will not be valid.

## After reading the notes above you may be wondering how to use your new TOPRO Troja 2G in a safe, ergonomic and comfortable way.

In order to ensure that the TOPRO Troja 2G is a good and safe aid for you, it is important that you keep the rollator close to your body. This is the best way to ensure good support while you are walking, and it will make it easier to get back up if you should have the misfortune of falling. Walking upright with your eyes gazing steadily forward will help you keep your balance.

**It is important that the height of handles are adjusted to best suit you. Please read point C on page 15 carefully to learn about handle adjustment. If you experience problems adjusting your rollator handles, please contact your dealer or physical therapist.**

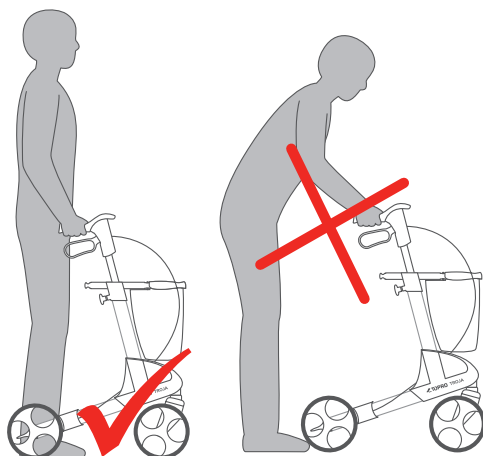## Warranty

TOPRO TROJA is guaranteed free of faults and defects for a period of 7 years. Our guarantee excludes damage caused by incorrect operation, accidents or through normal wear and tear (e.g. brake blocks, brake wires, wheels, seat, handgrips, basket and grips etc).

For repairs during the guarantee period, please contact your local mobility shop or dealer. The guarantee is void if unauthorised spare parts and accessories have been or are being used on the product.

## Expected lifespan of the product

The expected life span of the product is estimated to be 10 years, provided that it is used in the correct manner according to this user manual and its safety and maintenance instructions.

## Security and comfort

Where you find the symbols below, there is a special notice about the security or comfort of the product.

Comfort

Security

## Symbols on the TOPRO Troja 2G

Max 150kg

Maximum user weight

Max 610mm

Maximum rollator width

Max 1035mm

Maximum rollator height

A separate label on the tube indicates the serial number (SN) and production year and month.

## A. Folding and unfolding

Storage and travel


### Unfolding

To unfold the TOPRO Troja 2G, release the lock clamp from the carrying handles(1) and push them down until you can hear them click into place (2).

### Folding

When you want to fold your TOPRO Troja 2G, simply pull on the release strap on the seat (3). Then push the carrying handles together and lock them with the lock clamp (4). Make sure that the fabric of the front basket does not protrude too much between the carrying handles as that can block the fastening of the lock clamp.

Now your TOPRO Troja 2G is folded, still standing on all four wheels. It is now ready for transportation or storage. To reduce its size even further, it is possible to lower the handles (see on page 15, point C & D) and remove the wheels (see on page 16, point G).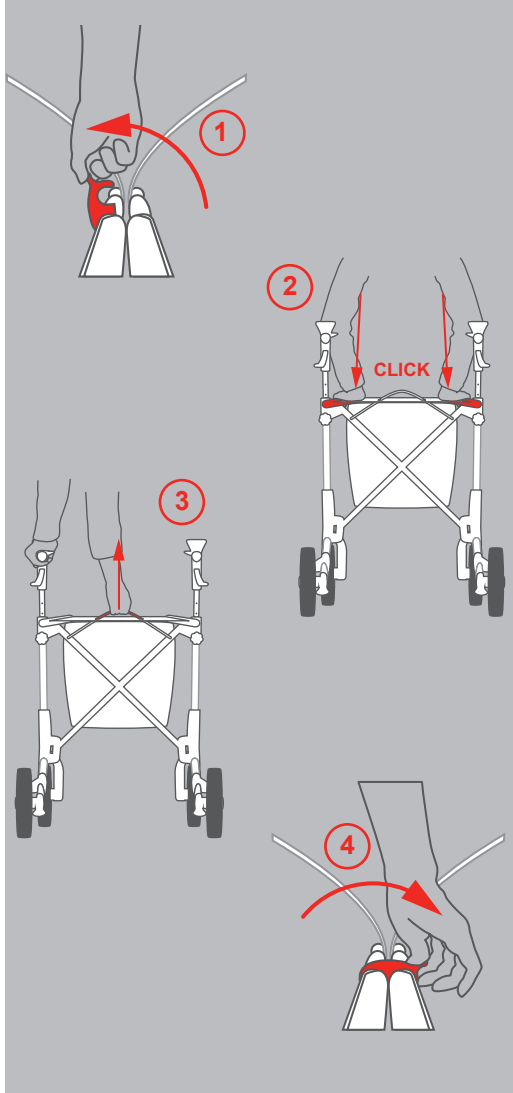

## B. Shopping basket

Transportation and shopping


Shopping basket (hereafter named basket) is enclosed. It holds up to 10 kilos and is easily attached and removed.

To mount the basket, stand between the handles of your rollator and apply the parking brake (1) Then lift the basket forwards in order to slide it underneath the seat (2), and onto the front and the carrying handles. When it is placed correctly, you will hear a click (3). Make sure that the brake wire still runs unrestrained along the frame (4).

To remove the basket, stand once again between the handlebars and apply the parking brake. Place your hands towards the front of the basket, one on either side. Lift the basket up while pressing it slightly together. This will release it. Always use both hands when performing this operation.

The shopping basket comes with a shoulder strap for you to carry it directly into your home. Make sure that the shoulder strap is tucked away when the shopping basket is fitted to the rollator, so that it does not become trapped in the front wheels.

The shopping basket can be washed gently at 40 °C.

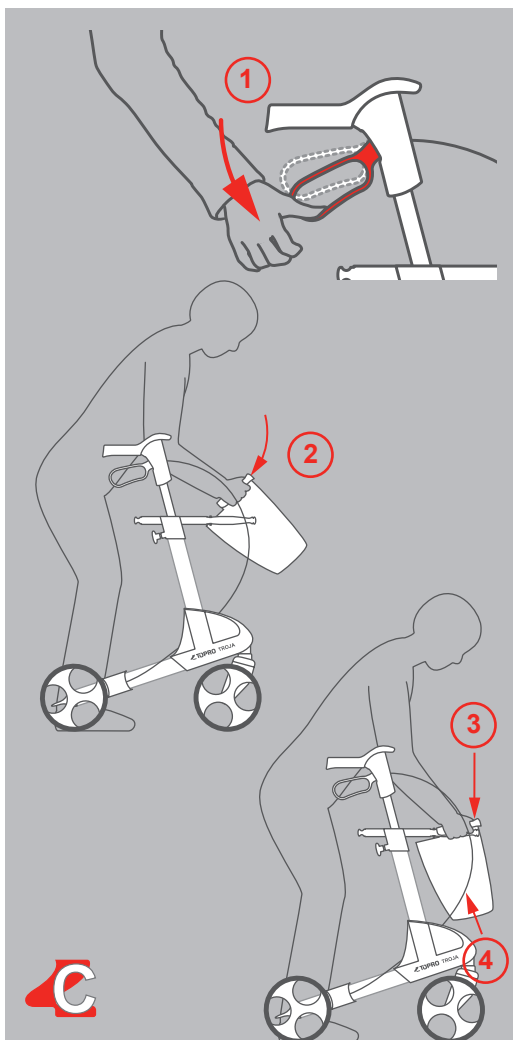### C. Adjusting the height of the handles


Walking and standing ergonomically correct

As mentioned before, it is very important both for your safety and well-being that the height of the handles are correctly adjusted. To find the correct height, stand between the handles of the TOPRO Troja 2G and let your arms hang loosely down with your elbows slightly bent and your palms turned inwards. Your wrist now marks the correct adjustment height of the handles.

Your TOPRO Troja 2G will support you safely if your handle height is correctly adjusted and you walk closely to it, as previously shown.

To adjust the handle height, release the lock wheel by turning it a few times to the left (1), adjust the handlebar tubes to a suitable height (2) and pull them gently up and down until you hear a click. Then tighten the lock wheel again (3).

It is important to adjust both handles to the same height in order to maintain body symmetry and correct posture.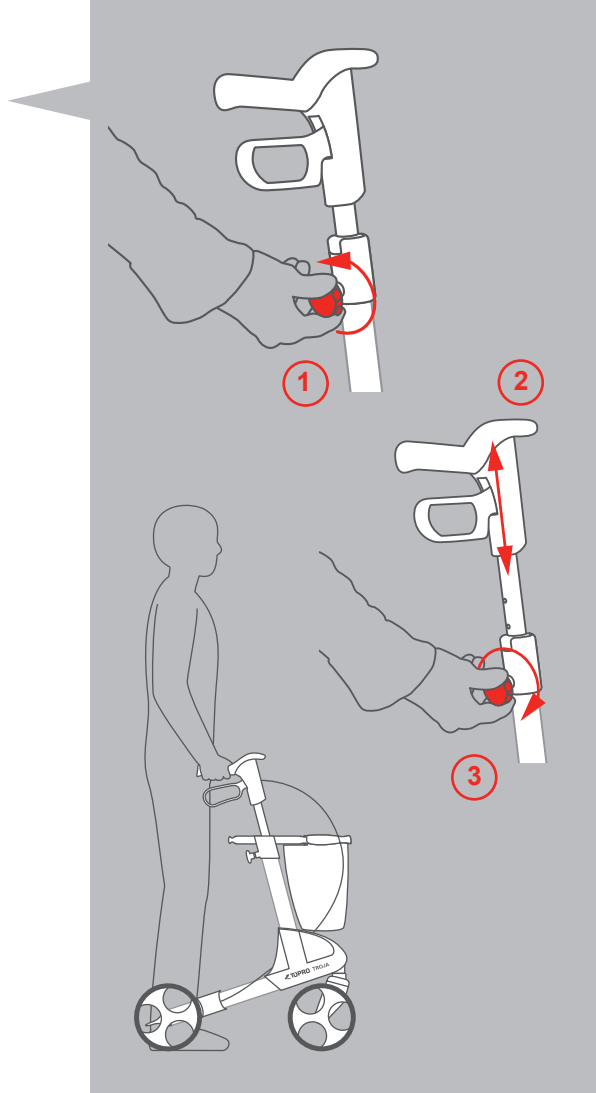


### D. Height adjustment using the memory function

Always at the correct handle height

You have the opportunity to set the height of the handles at a standard setting, fitted to your requirements for your height, so that you easily can find the same height again. This can be useful when the rollator is being transported or stored as compact as possible.

Decide the height of the handlebars (see C, *Adjusting the height of the handles*). Count visible holes at the back of the handlebar tubes. Unscrew the lock wheel (1). Pull the tube out of the frame (2). The attached part will now follow the handlebar tube. Release the red memory button by turning it halfway round and pulling it out (3). Count down to the correct hole on the handlebar tube as previously determined and insert the memory button there. Then place the handlebar back into the frame (4) and tighten the lock wheel (5).

Now, to adjust the grip height, simply loosen the lock wheel (1) and pull up the handlebars as far as they go (2). Tighten the lock wheel (5). You have now set your optimal ergonomic grip height. Hereafter your handlebars will automatically stop at the correct height when you pull them up.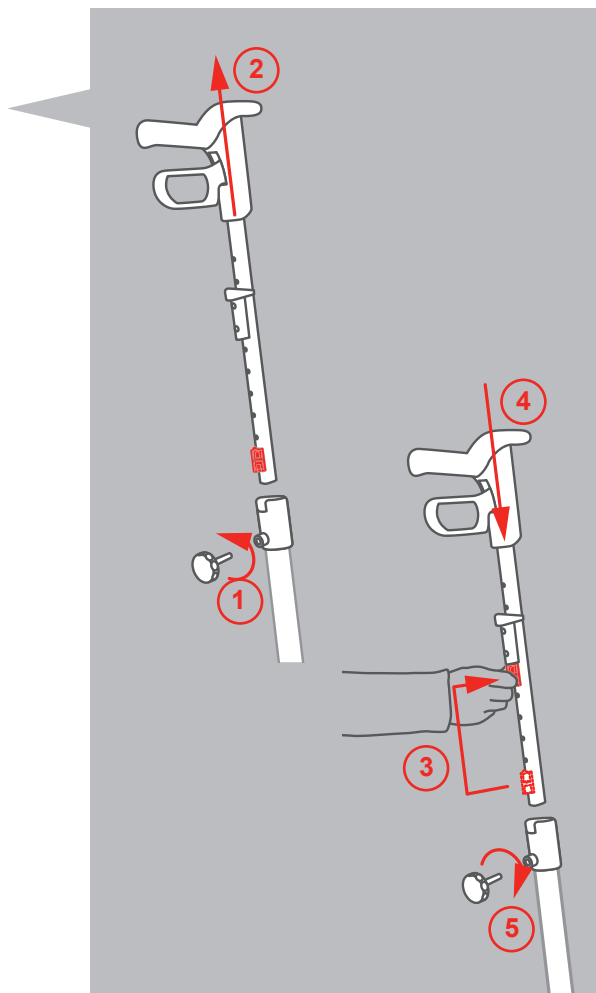


## E. Brakes

### Stopping and parking

The brake handle can be used in two different ways  
- as a driving brake or as a parking brake.

**Driving brake (1):** The driving brake is used to reduce your speed when pushing the TOPRO Troja 2G. To apply the driving brake, simply pull the brake handles up. The brakes are applied instantly. Release the handles, and the TOPRO Troja 2G is once again ready to move forward. To prevent damaging the wheels, the brakes should not be applied continuously while walking.


**! Parking brake (2):** The parking brake is used when TOPRO Troja 2G is parked for a short or long period, i.e. when you want to take a break from walking and sit on the seat. It is vital to apply the parking brake in this situation! To apply the parking brake, push the brake handles down towards the frame. To release, simply pull them back up.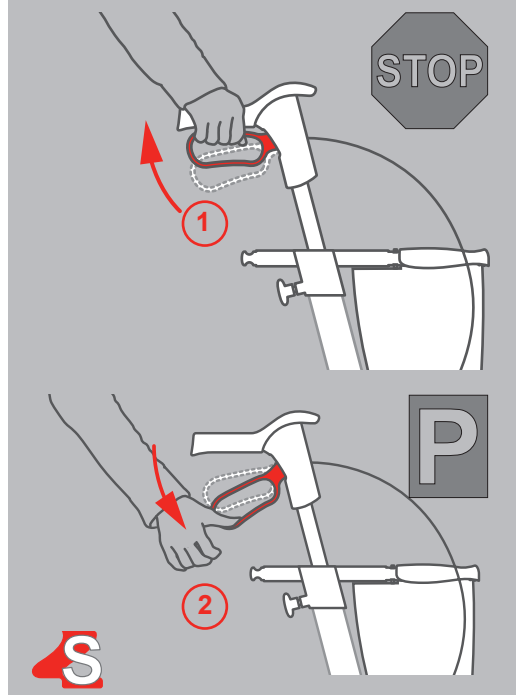

## F. Adjusting/checking and maintaining the brakes

Loosen the nut beside the brake (1). Tighten the brake wire with the adjustment screw (2). Adjust the brake block (3) as close as possible to the inner drum brake, and turn the wheel around to check that the brake block is not touching the wheel.

The brake blocks and springs should be cleaned regularly to guarantee their optimal operation. The wheels and brakes can be replaced if they are worn or damaged.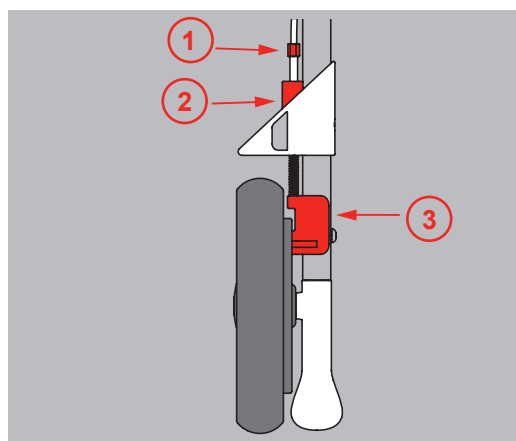

## G. Replaceable wheels "TOPRO Quick"

### Suitable wheels for any surface


If you want to remove the wheels for transportation or replacement, follow these instructions:

**Remove:** Loosen the brake lever. Push down the red button on the hub (1), and slide the wheel off the axle while still pushing the button down.

**Attach:** Loosen the brake lever. Push down the red button on the hub (1). Slide the wheel onto the axle while keeping pressure on the button (2). Push the wheel as far as possible inward on the axle and release the button to make the wheel snap onto the axle. The wheel is securely fastened when the axle is completely visible in the middle of the hub and you cannot see any of the red marks (3). The red button should now have popped back up from the hub. Gently pull the wheel to check that it is securely fastened to the axle.

**After the new wheel has been applied to the rollator, make sure that the brake lever is reattached and the brakes are functioning properly. Readjust them if necessary (p.16, point F).**

In addition to the standard wheels there are also offroad wheels and studded tyres available as accessories. This is possible due to the fact that the rollator has got IBS (interior braking system) which brakes on the rim instead of the wheel surface.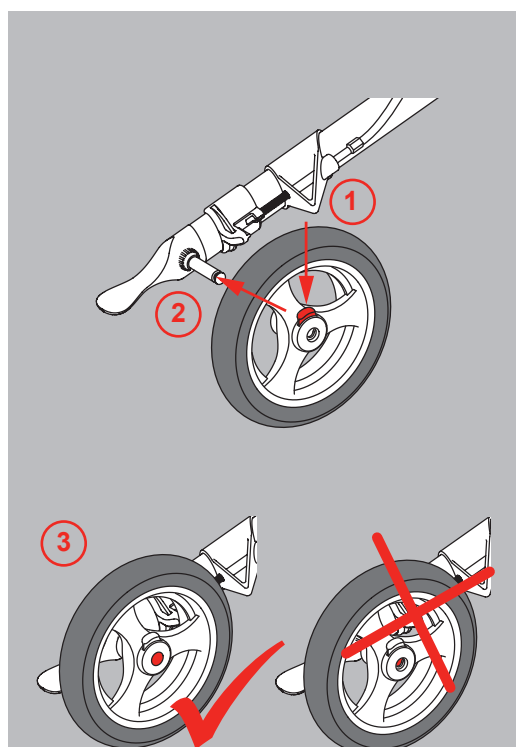


## H. TOPRO Ergo Grip

Comfortable support when rising, standing or walking

TOPRO Ergo Grip offers new ergonomical grip and support surfaces. It combines the original hand grip position of the TOPRO Troja models (1) with a hand grip (2) at the front of the handle. This gives you a better holding position when getting up from a chair or couch (3). Always make sure the parking brake is locked in when you do this (4). The new hand grip also offers extra support while standing (5).

Several options of support makes it easy to vary your position when standing, thereby increasing your comfort.

When pushing the rollator your hands should always be kept in the original grip position (1) in order to make the brakes accessible.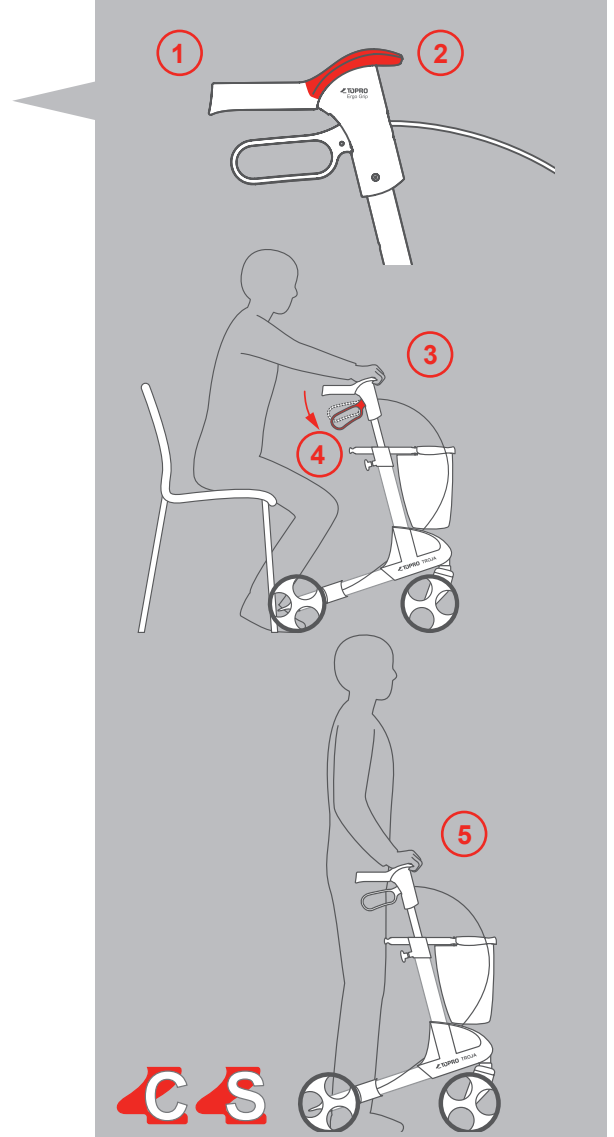


## I. Seat

Sitting and relaxing

When you want to relax for a while, you can sit comfortably on the seat of TOPRO Troja 2G. Make sure to apply the parking brakes (1), to stop the rollator from moving! Always sit down with your back turned towards the shopping basket (2)!

For your own safety, do not bend backwards or lean over to the sides when sitting on the rollator (3).


A back support is available as an accessory to provide additional comfort when sitting on the rollator.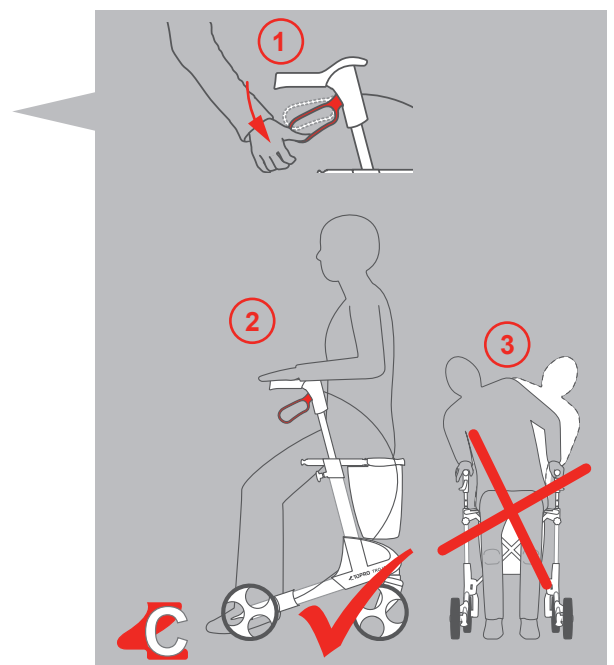

## J. Edge guard

Safety while walking

The edge guard (1) on the rear wheel prevents the rollator from catching onto protruding edges, door frames etc. It allows you to manoeuvre safely past such obstacles.

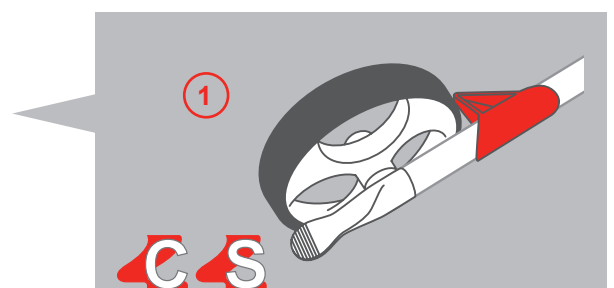
## K. Tilt function

### Getting past obstacles

The wide tilt lever (1) helps you to navigate kerbs, thresholds and other minor obstacles. Simply place the front wheels towards the obstacle, place one foot on the tilt lever and press down (2). The TOPRO Troja 2G will then tilt up at the front and can be safely pushed across the obstacle.

**!** Please note: Take care when descending kerbs. Do not use the rollator in stairs.

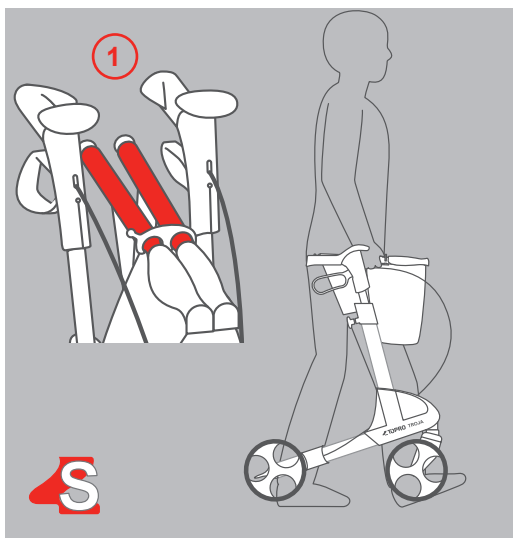## L. Carrying the rollator

### Transportation and storage

If you wish to carry your TOPRO Troja 2G, please do so using the carrying handles (1).

This is the only grip you should use for this purpose. Carrying the rollator by the brake wires or seat strap may cause harm to the rollator or the user!

The carrying handles are also used to fasten the shopping basket.

## Storage

Store the TOPRO Troja 2G indoors, in an upright position. It can also be stored outdoors, but it is recommended that it is placed in a sheltered position out of the weather. To help prevent any damage to your TOPRO Troja 2G we recommend that it is stored under cover and away from extremes of hot and cold temperatures.

## Transport

The rollator should be transported in the folded and locked position (see paragraph *Folding and unfolding*). Ensure that the rollator cannot unfold unintentionally. Be careful when loading the rollator into a vehicle, giving attention to securing of the item.

## If the rollator does not work properly

Make sure the rollator is unfolded correctly and that both brake handles are in expected position.

## Technical data

### TOPRO Troja 2G

Size	M (medium)	S (small)
Recommended user height	150–200 cm	135–170 cm
Max. user weight	150 kg	125 kg
Max. capacity basket	10 kg / 15 l	10 kg / 15 l
Handle height	780–1000 mm	670–860 mm
Maximum length	650 mm	650 mm
Maximum width	600 mm	600 mm
Distance between handles	470 mm	470 mm
Seat height	620 mm	540 mm
Turning diameter	840 mm	840 mm
Diameter front/rear wheels	200 mm	200 mm
Width front/rear wheels	36 mm	36 mm
Handles	Ergonomic – several holding positions (TOPRO Ergo Grip)	
Grip distance driving brake	65 mm	65 mm
Dimensions when folded	l 650 × w 240 × h 815 mm	l 650 × w 240 × h 740 mm
Transport dimensions (without wheels and basket)	l 600 × w 240 × h 760 mm	l 600 × w 240 × h 720 mm
Transport weight (without wheels and basket)	5.5 kg	5.1 kg
Weight with basket and wheels	7.8 kg	7.4 kg

### Inspection / Maintenance / Re-use

It is recommended that general inspection and maintenance is carried out and that the items below are checked on a regular basis, how frequent is determined by how often and how the rollator is used:

- Tubes, frame and screws
- Handlebars and handlebar tubes
- Brakes and brake handles, -blocks and -springs
- Wheels, seat and accessories

This also applies when the rollator is to be prepared for re-use. Please see detailed instructions in this user manual about maintenance of the brakes and cleaning and disinfection of the rollator.

### Cleaning

If your TOPRO Troja 2G has become dirty after use, simply wash or wipe it with normal household detergents. Do not use scrub detergents, steam- or high-pressure cleaners.

### Disinfection

Disinfection must only be carried out by authorized personnel and by wearing sufficient protective equipment.

The surface of the rollator must be cleaned with a disinfection that contains 70-80% ethanol. We advise against disinfection that contains chlorine and phenol. *The manufacturer cannot be held responsible for any damage or wounds that might occur by the use of harmful disinfection or disinfection carried out by unauthorized personnel.*

### Material / Recycling

The rollator is made of plastic coated aluminium pipes, plastic couplings and material of plastic and polyester. Most parts can be recycled. Dispose the rollator according to the applicable regulations in your country. For information, please contact your public authorities.

### TOPRO Accessories

TOPRO has developed a wide range of accessories following feedback from our satisfied customers. Function, design and quality are key words in these developments, and we aim to provide you with smart and innovative solutions for your everyday life with your new TOPRO Troja 2G. You will find a selection of accessories described on the next pages or on [www.topro.co.uk](http://www.topro.co.uk) and at your TOPRO dealer.

**NEW!**  
**Offroad Wheels** for IBS.  
 Good grip when walking on rougher paths etc. Set of 4.  
 Article no. 814649

**NEW!**  
**Studded Tyres** for IBS.  
 Good grip when walking on icy roads etc. Set of 2 rear wheels.  
 Article no. 814670

**Ice spurs for shoes**  
 Size M (ca. 36–40)  
 Size L (ca. 41–46)  
 Article no. 814029 (Size M)  
 Article no. 814035 (Size L)


**Adjustable Back support with padding**  
 Ergonomically adjustable depending on your height and preferred seating position

Article no. 814607


**Tray**  
 an easy way to carry cups and plates etc.  
 Easy to remove and clean

Article no. 814728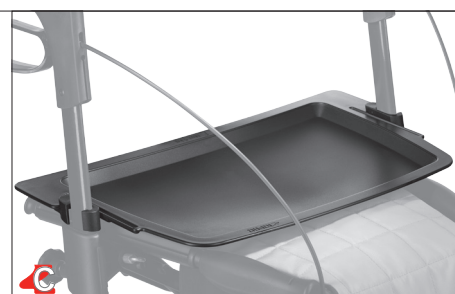


**Anti slip mat for tray**

Article no. 814059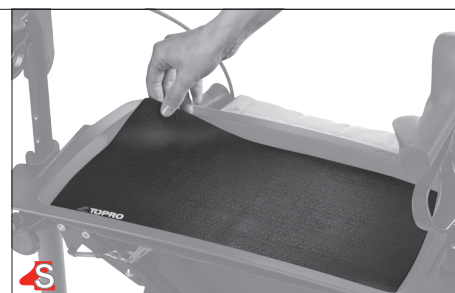

**Crutch holder**  
 can be mounted on either side

Article no. 814025

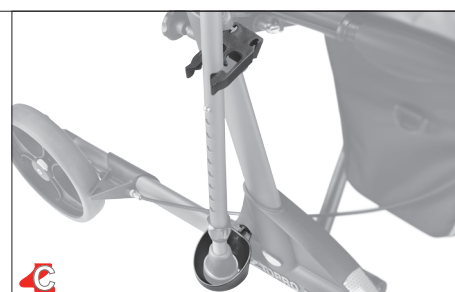**Bottle holder**  
 so that you won't get thirsty  
 (Bottle not included)

Article no. 814043

**Rollator lock**  
 additional security for your TOPRO Troja 2G

Article no. 814023

## TOPRO Original accessories

### LED lamp

Light gives security in the dark

Article no. 814608

### Rear bag with zipper

Keep your personal belongings like wallet and mobile phone in a practical bag close to you.

Article no. 814046

### Nameplate

avoid mix ups and be sure you have your own TOPRO Troja 2G

Article no. 814024

### One handed brake

Can be mounted on either side


Article no. 814026

### Guiding handle

e.g. for the visually impaired

Article no. 814014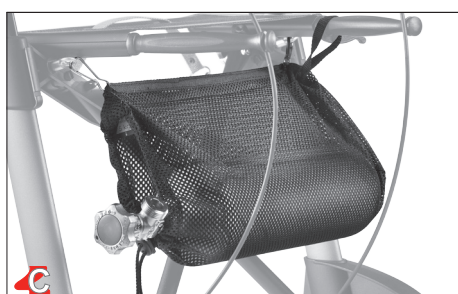

### Oxygen basket

suitable for bottles with 2 litres of oxygen (only compressed, not liquid)

Article no. 814009

### Umbrella with attachment

Keep dry under the umbrella in rainy weather or use it as a parasol.

Article no. 814068

### Decorative stickers (set of 4)

Article no. 814141

**Transport bag** protects your rollator when travelling by e.g. airplane, train and car

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TOPRO GmbH
TOPRO Limited