**TOPRO
TROJA NEURO**

TOPRO

Made in Norway

NO	Bruksanvisning	3–6
EN	User manual	7–10
DE	Bedienungsanleitung	11–15
FR	Manuel d'utilisation	16–20
NL	Gebruiksaanwijzing	21–25

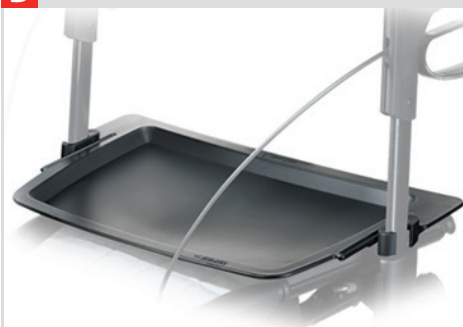
www.topro.no

CE

A 814607

B 814728

C 814059

D 814046

E 814622

814672

814643

814641

F 814025

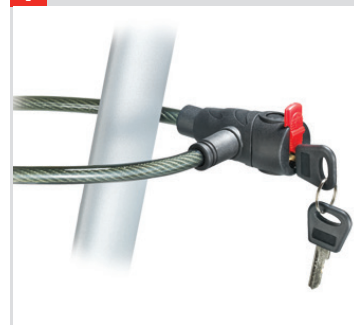
G 814608

H 814024

I 814023

J 814141

K 814043

L 814068

M 814042

N 814026

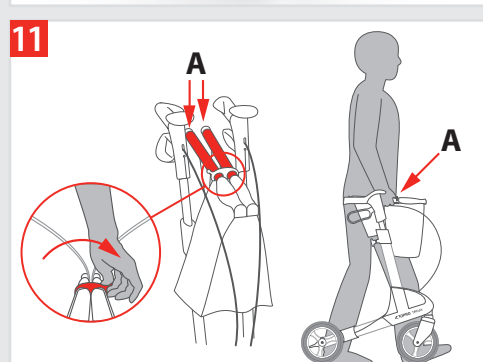
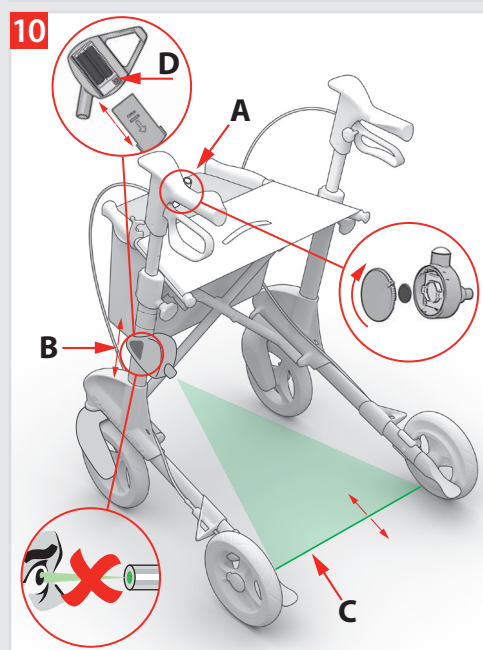
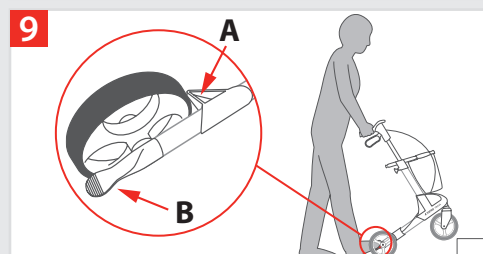
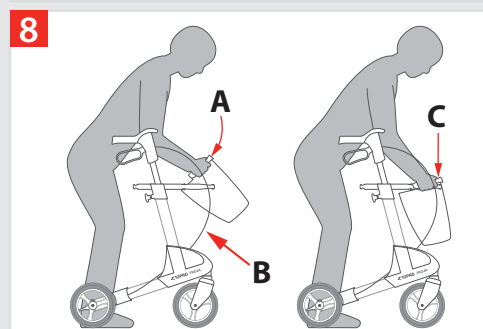
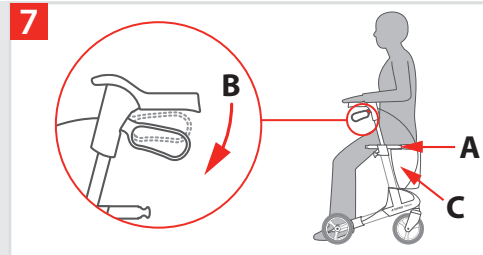
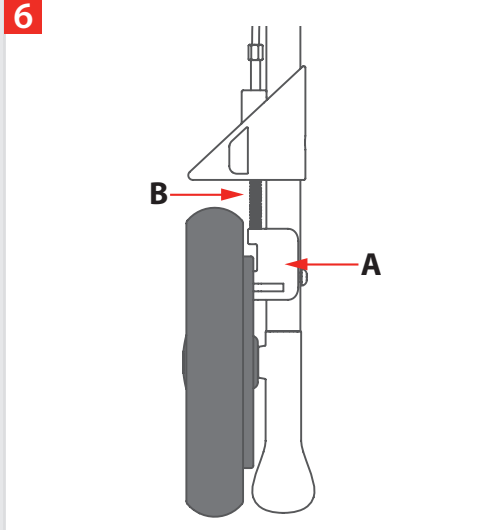
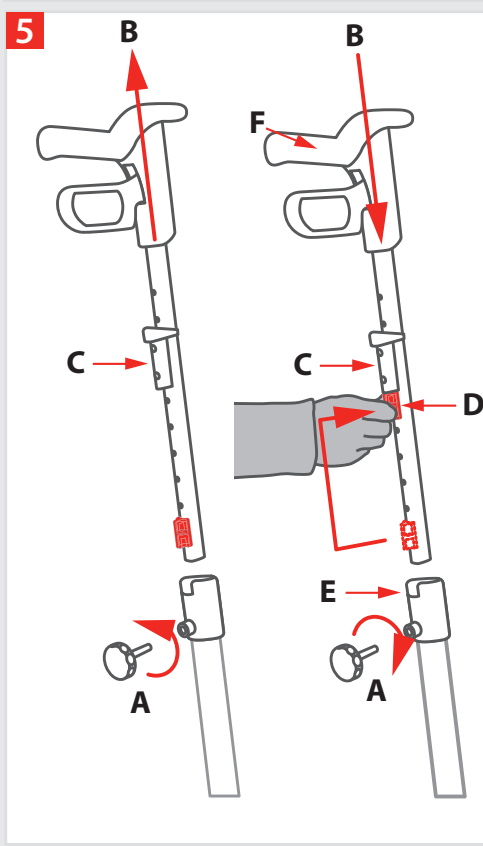
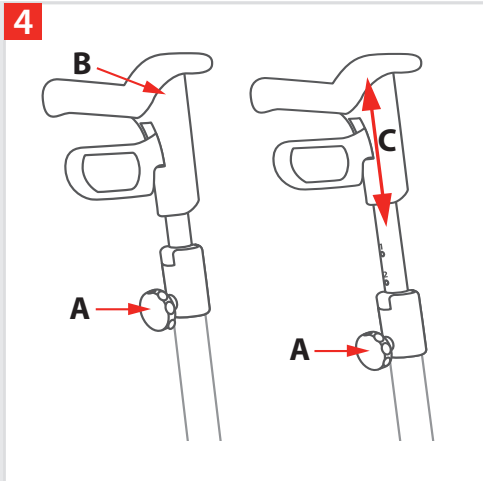
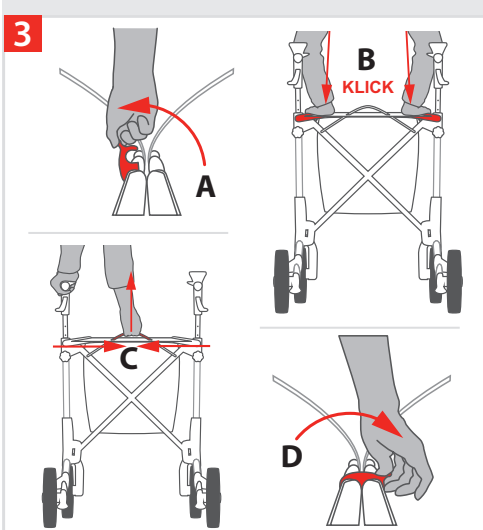
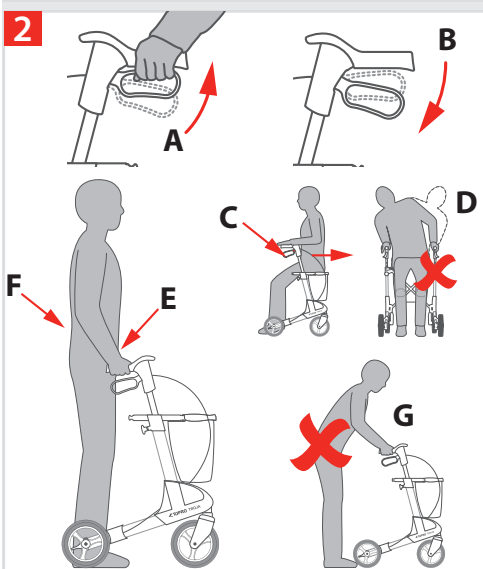
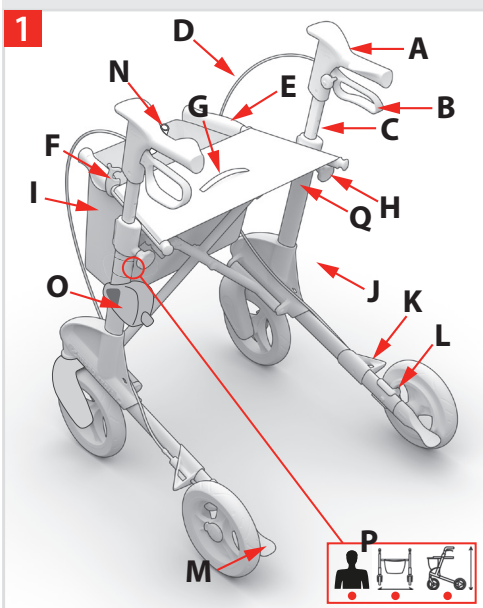
O 814014

P 814009

Standard model size M **814746**
 Standard model size S **814747**
 Laser model size M **814743**
 Laser model size S **814744**

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Welcome as a user of a TOPRO PRODUCT

Every TOPRO product is developed, designed and manufactured in Norway. Due to advanced manufacturing by using the best equipment, we are able to offer great accuracy and maximum quality in all levels of our production. This product conforms to the Medical Device Directive 93/42/EEC.

On receipt of the goods, please check your product, if you have any queries please contact your mobility shop or dealer immediately. The same applies if you have general questions about the product. If you have problems reading this user manual, an electronic version is available on www.topro.no

Guarantee // Expected life span of the product

TOPRO TROJA NEURO is guaranteed free of faults and defects for a period of 7 years. Damage caused by incorrect use or parts that are exposed to natural wear and tear (e.g. brake block, brake cable, wheel, seat, basket, handgrip, laser light and switch) are exempt from the 7 year guarantee. For repairs during the guarantee period, please contact your local mobility shop or dealer. The guarantee is void if unauthorised spare parts and accessories have been or are being used on the product.

The expected life span of the product is estimated to be 10 years, provided that it is used in the correct manner according to this user manual and its safety and maintenance instructions.

■ The red numbered squares in headings refer to the numbered illustrations on the inside of the cover.

Get to know your TOPRO PRODUCT 1

- | | | |
|------------------------------------|--------------------------------------|--|
| A Handlebar | F Lock clamp | K Edge guard |
| B Brake handle | G Seat with strap for folding | L Brake block |
| C Handlebar tube | H Lock wheel for the handle | M Tilt function |
| D Brake wire | I Basket | N Switch for laser light (option) |
| E Seat bars/carrying handle | J Side frame | O Laser light (option) |
- P** The label on the rear side of the left tube shows the product name/model and
 ■ Maximum weight of user ■ Maximum width of rollator ■ Maximum height of rollator
- Q** The label on the rear side of the right tube indicates the serial number (SN) and production year and month.

Measurements	TOPRO TROJA NEURO M	TOPRO TROJA NEURO S
Recommended user height	1500–2000 mm	1350–1700 mm
Maximum weight of user	165 kg	125 kg
Maximum capacity basket	10 kg / 16 l	10 kg / 16 l
Handle height	780–1000 mm	670–860 mm
Maximum length	650 mm	650 mm
Maximum width	600 mm	600 mm
Width between handles	470 mm	470 mm
Seat height	620 mm	540 mm
Turning diameter	840 mm	840 mm
Diameter wheels	200 mm	200 mm
Width wheels	36 mm	36 mm
Handles	Ergonomic – several holding positions (TOPRO Ergo Grip)	
Grip distance driving brake	65 mm	65 mm
Height when folded	800 mm	740 mm
Width when folded	240 mm	240 mm
Length when folded	650 mm	650 mm
Product weight	7.2 kg	7.0 kg

Intention

The TOPRO TROJA NEURO shall give support to users with reduced balance and/or reduced walking ability. It is developed to be pushed, not pulled. The rollator has a reverse brake system. The brake is activated at all times and released only when the brake handles are used. The aim is to increase the user's ability to maintain control. The TOPRO TROJA NEURO can be delivered with a laser light unit, which may be of help for persons with a neurological disease and suffer from "freezing" episodes. The integrated seat provides an opportunity for the user to rest if needed. The fabric basket is designed for carrying items. The product is designed and approved for indoor and outdoor use. For outdoor use, the rollator is intended to be an aid for walking on roads and paths. It is not suitable for rough and/or very steep terrain, neither for stairs. The product is not suitable for persons with low strength in arms, with very poor balance, or with considerable cognitive disabilities.

Important to know about safety before use

- Take care when folding and unfolding to ensure not to injure your hands or fingers.
- Check the brakes before use, and that the rollator locks into position when opened.
- Not to be used by persons heavier than the maximum user weight (see table).
- Do not use to transport heavy loads or persons.
- The rollator is not a toy and therefore should be treated in an appropriate manner.
- Make sure that the lock wheel for height adjustment for the handles is locked tightly.
- Be aware! Some parts of the rollator might feel hot or cold when exposed to extreme temperatures.
- Do not alter the product as this might put your safety at risk and the guarantee will not be void.

Important to know about safety when in use **2**

- In order to move the rollator, the brake handles must be pushed in **A**.
- In order to brake, release the brake handles. This activates the parking brake **B**.
- The parking brake is always activated, when you do not push in the brake handles, to prevent the rollator from rolling away **C**.
- Do not lean or bend sideways or over the back of the rollator whilst in the seated position **D**. Do not tilt the rollator more than max. 5° sideways, to avoid risk of falling. Due to the same reason bags etc. must never be hung on the handlebars.
- To give the right support and safety, make sure the rollator is kept as close to your body as possible **E**.
- Walk straight, look forward and hold your backside in **F**. That is the easiest way to keep your balance.
- Do not push the rollator too far from your body as this can cause unnecessary strain and you could fall **G**.
- Do not use the rollator on stairs. Be careful when descending kerbs.
- Be careful of flared skirts and trousers as they could get caught in the wheels, which could lead to falling. We therefore recommend not wearing flared/wide clothing.
- For safety reason it is prohibited to sit on the rollator during public transportation. Place your rollator securely.
- NB! For rollator models with laser light unit: Do not look directly towards the laser beam, as this may cause injuries on your eyes.

Assembling / adjusting

TOPRO TROJA NEURO is delivered assembled. Before use, you must unfold the rollator. The height of the handlebar tubes will then need adjusting to suit the user's needs.

Unfolding / folding **3**

Unfolding: Loosen the lock clamp that is over the seat bars **A**. Press the seat bars down and out **B**. The side frames will slide apart. Press down the seat bars. You will hear a clicking sound. The rollator is now locked in its open position.

Folding: Pull the strap on the seat **C** and press the side frames together. Lock the clamp **D** over the seat bars.

Lowering and height adjusting the handles **4**

Lowering: Unscrew the lock wheel for the brake handles **A** (approx. 4–5 turns). Push the handlebar tubes down until they stop **B**. Tighten the lock **A** again.

Height adjusting: Unscrew the lock for the brake handles **A** (approx. 4–5 turns). Adjust the handlebar tubes to a suitable height then pull them slightly up and down **C** until you hear a clicking sound. Tighten the lock **A** again. Adjust both handlebars to the same height. *It is recommended that the handlebars are set to the same height as your wrists when your arms are hanging alongside your body.*

Height adjustment using the memory function **5**

You have the opportunity to set the height of the handles at a standard setting fitted to your requirements so that you easily can find the same height again. Decide the height of the handlebars. Count visible holes at the back of the handlebar tubes. Unscrew the lock wheel **A**. Pull the tube **B** out of the frame. The attached part **C** will now follow the handlebar tube. Move the attached part **C** on the tube **B** so the amount of holes corresponds to the amount counted earlier. Fit the memory chip **D** on the handlebar tube in the first available hole under the attached part **C**. Push the handlebar tube **B** down and into the side frame **E** until the attached part is locked. Screw the lock wheel **A** half in. Pull the handlebars **F** up until they stop and then tighten the lock wheel **A** properly. From now on, when the handlebars are in the lowest position, they will stop in the preset position automatically when pulled up.

Using the brakes **2**

Make sure the brakes work before each use and that the rollator moves when the brake handles are pushed in **A**. Each brake handle **B** works on each of the rear wheels.

The rollator wheels are made of a special soft rubber, which gives an optimal braking capacity on slippery floor. Wrong use can cause unsafe situations and damage to the rollator: *The rollator must not be driven or pushed with force when the brake handles are not pushed in.* This indicates incorrect use and can cause wear and tear, which makes the tires, and brake quality weaker. **Parking brakes:** Release the brake handles **B**. Both rear wheels are now locked. Squeeze both brake handles upwards to release the parking brakes **A**. To brake during walking, release the brake handles. *Tip: If the handles are pushed all the way down to lock position, the rollator can move freely.*

Maintaining the brakes **6**

Clean the brake block **A** and the brake spring **B** frequently. The wheels and brakes can be changed if they are worn out or damaged.

Seat **7**

The brake handles **B** must be released before using the seat **A**. Both rear wheels are now locked. Sit with your back towards the basket **C**.

Basket **8**

The basket is enclosed. It holds up to 10 kilos and is easily attached or removed.

To mount the basket, stand between the handles of your rollator and make sure that the brakes are on. Then lift the basket forwards in order to slide it underneath the seat and onto the front and the carrying handles **A**. When it is placed correctly, you will hear a click. Make sure that the brake wire still runs unrestrained along the frame **B**. To remove the basket, stand between the handlebars and place your hands towards the front of the basket, one either side. Lift the basket up while pressing it slightly together. This will release it **C**. Always use both hands when performing this operation. The bag can be washed gently at 40 °C.

Edge guard // Tilt function **9**

Edge guard: The edge guards **A**, which are in front of the rear wheels prevent the rear wheels from catching on corners, door frames etc.

Tilt function: To use the tilt function use the front of the foot and press down on the tilt pedal **B**, which is beside the rear wheel. This helps to tilt the rollator over thresholds, edges etc. **Please note:** Take care when descending kerbs. Do not use the rollator in stairs.

Laser light **10** (applies to the models 814743 and 814744)

The TOPRO TROJA NEURO can be delivered with a laser light that can be of help for users with neurological disease that suffer from "freezing" episodes. The laser light can easily be operated from the handle and the transfer is wireless to the laser light box.

Batteries must be inserted before the laser light can be used: Unscrew the lid **A** located by the left-hand handle. Insert the flat battery (Lithium CR 2032) and screw the lid back on. Thereafter, insert two AA batteries (LR6) in the laser light box located on the left frame bar **B**. The lid, located on the inside, must be pushed down. Make sure to follow the "+/-" signs when inserting the batteries. The batteries cannot be recharged and will normally last for approximately 1 year with normal usage (approximately 15 minutes per day). Beware that the battery lifetime will decrease as the frequency and length of usage increase.

Use of laser light: Push the button by the left-hand handle **A**. The laser light unit **B**, which is located on the left frame tube, will then generate a green line in front of the feet of the user **C**. The laser light unit is designed to withstand humid weather but is not 100 % waterproof. Caution! Both boxes **A** and **B** must be removed during hosing or disinfection of the rollator.

NB! This Class 3R-laser is considered safe for normal operation. However, we advise you not to look directly towards the laser beam from the laser light unit **B**, to protect your eyes from injuries. **Laser light settings:** Moving the laser light unit up or down the frame tube can move the location of the laser beam forward or backward. The amount of seconds/minutes the laser beam is activated as a result of pushing the laser light button **A** can be regulated by adjusting the switch **D** located inside the laser light unit **B**. Push down the lid and adjust the switch with a screwdriver in the positions 1–4, corresponding to the following durations for the laser beam:

1 = 15 seconds 2 = 30 seconds 3 = 120 seconds 4 = flashes for 15 seconds

Carrying the rollator **11**

The rollator must not be carried by the seat strap or the brake wires. Fold the rollator together as shown on the illustration. Carry the rollator by the folded seat bars **A**.

Transport

The rollator should be transported in the folded and locked position (see paragraph *Unfolding / folding*). Ensure that the rollator cannot unfold unintentionally. Be careful when loading the rollator into a vehicle, giving attention to securing of the item.

Storage

The rollator should be stored in an upright position. Models with laser light must be stored inside.

Material // Recycling

The rollator is made of plastic coated aluminium pipes, plastic couplings and material of plastic and polyester. Most parts can be recycled. The laser light and the switch for the laser light should be recycled as electronic waste. Dispose the rollator according to the applicable regulations in your country. For information, please contact your public authorities.

Cleaning

The rollator can be steamed, hosed and washed with normal household detergents. Do not use scrub detergents, or high-pressure cleaners. For models with laser must the laser light unit and the switch box be removed during steaming or hosing.

Disinfection

Disinfection must only be carried out by authorized personnel and by wearing sufficient protective equipment. The surface must be cleaned with a disinfection that contains 70–80 % ethanol. We advise against disinfection that contains chlorine and phenol. The laser light unit and the switch box must be removed during disinfection. *The manufacturer cannot be held responsible for any damage or wounds that might occur by the use of harmful disinfection or disinfection carried out by unauthorized personnel.*

Inspection // Maintenance // Re-use

It is recommended that general inspection and maintenance is carried out and that the items below are checked on a regular basis (how frequent is determined by how and how often the rollator is used): Tubes, frame and screws, handlebars and handlebar tubes, brakes, wheels, seat, laser light and accessories. This also applies when the rollator is to be prepared for re-use. Please refer to the detailed instructions presented in this user manual about cleaning, disinfection, and battery replacement.

If the rollator does not work properly

Make sure the rollator is unfolded correctly and that both brake handles are in expected position.

Accessories

For pictures, please see the inside of the cover of this user manual. Accessories mounted on the rollator may influence stability, we therefore advise you to use them carefully.

- | | |
|------------------------------------|--|
| A Back support with padding | I Lock |
| B Tray | J Flower stickers, set with 4 stickers |
| C Anti slip mat for tray | K Bottle holder |
| D Rear bag with zipper | L Umbrella (black) |
| E Shopping bag | M Transport bag |
| F Crutch holder | N One-handed brake |
| G LED lamp | O Guiding handle for e.g. visually impaired |
| H Nameplate | P Basket for oxygen bottle (compressed, not liquid) |

The accessories are to be ordered separately to give the possibility to compose a TOPRO TROJA NEURO that suits the individual user's needs. Contact your mobility shop, dealer or TOPRO for an updated overview of the accessories, or visit our homepage www.topro.no

In case of questions or product faults, please contact your mobility shop or dealer.

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